



**The First Aid  
Kit in a Bottle**



**For Use On ALL  
Animals Including  
Birds & Reptiles**

**NET CONTENT:  
1 FL OZ (30 ML)**

**LIMITED WARRANTY AND DISCLAIMER**

The manufacturer warrants only that the chemical composition of this product conforms to the ingredient statement given on the label and that the product is reasonably suited for the labeled use when applied according to the Directions for Use.

THE MANUFACTURER NEITHER MAKES NOR INTENDS ANY OTHER EXPRESS OR IMPLIED WARRANTIES, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, WHICH ARE EXPRESSLY DISCLAIMED. This limited warranty does not extend to the use of the product inconsistent with label instructions, warnings or cautions. Any damages arising from a breach of the manufacturer's warranty shall be limited to direct damages, and shall not include indirect or consequential damages such as loss of profits or values, except as otherwise provided by law.

The terms of this Limited Warranty and Disclaimer cannot be varied by any written or verbal statements or agreements. No employee or agent of the seller is authorized to vary or exceed the terms of this Limited Warranty and Disclaimer in any manner.

**For more information call  
1-800-336-7087  
or visit [www.emtgel.com](http://www.emtgel.com)**

245/3-2005

**INGREDIENTS:**

Collagen, Purified Water,  
Sodium Benzoate, Bitrex™.

MANUFACTURED FOR  
**TROPHY**  
ANIMAL HEALTH CARE  
PENSACOLA, FLORIDA 32514  
[www.trophyanimalcare.com](http://www.trophyanimalcare.com)

**FOR ANIMAL USE ONLY**

**READ CAREFULLY BEFORE USING**

EMT Spray is a revolutionary breakthrough in wound care for animals. EMT Spray provides nutritive collagen protein to the wound site to enhance the natural healing process and promote rapid healing. EMT Spray seals off nerve endings to reduce pain and itching. EMT Spray also forms a semi-occlusive barrier that protects the wound and provides a moist environment which encourages healing. EMT Spray also contains Bitrex™ to deter licking and biting of wounds.

**INDICATIONS —**

For topical application on all animals in the management of chronic and acute wounds such as:

- Hot Spots
- Superficial wounds (cuts, abrasions)
- Burns
- Traumatic wounds (flesh tears)
- Injured foot pads
- Surgical wounds
- Other general dermatologic conditions

**CHARACTERISTICS —**

EMT Spray reduces the chance of infection by forming a barrier over the wound site.

Easy to apply, EMT Spray:

- Promotes Rapid Healing
- Reduces biting/licking of wounds
- Reduces Pain and Itching
- Protects Wounds
- Reduces Bleeding
- Conforms to any Wound Site
- Soothes and Deodorizes
- Absorbs Wound Fluids
- Biocompatible, Biodegradable

**DIRECTIONS FOR USE:** Cleanse the site. Shake spray bottle well. Hold spray bottle 4-6 inches away from the area to be treated. Spray a light coating directly onto the wound and surrounding area. Allow to air dry; cover large wounds with a non-stick dressing. Secure the non-stick dressing if needed. Remove dressing as needed (if the dressing sticks to the wound soak in warm water to ease removal).

**PRECAUTIONS:** Keep out of reach of children. Avoid contact with the eyes as the Bitrex may cause eye irritation. Avoid contact with the mouth and nose, as the Bitrex is very bitter. If condition worsens or does not improve within 10-14 days, contact your veterinarian.

**STORAGE:** Store at room temperature 15°-30°C (59-86°F).



Bitrex is a trademark of Macfarlan Smith.